CUSTOMER :	VISCO TECHNOLOGY SDN. BHD.
DESCRIPTION :	44.0306C.522 BMC Miacare DELIGHT CN&EN&MYMonthly SiHyVer.3
MEASUREMENT :	195MM X 276MM
B/QUALITY :	SIMILI 060GSM

SHEET SIZE :	N/A	COLOUR:	
PROCESS :	DIE-CUT / FOLDING (MACHINE)		
FINISHING :	N/A		
MOULD CODE :	N/A		

K(B)

Date:

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CAUTION

This Device to sale by or on the order of a licensed eye care professional

PRODUCT DESCRIPTION

Water Contents	Material	Base Curve	Diameter	Power Range	Coating Color
47%	NVP, Siloxane macromer	8.2~9.0mm	13.8~14.4mm	-12.00D to +8.00D (-12.00~-6.00,0.50/step, -6.00~+6.00,0.25/step, +6.00~+8.00,0.50/step,	Blue

INDICATIONS (USES)

miacare® DELIGHT Monthly Silicone Hydrogel Soft Contact Lens is indicated for the optical correction of refractive ametropia (myopia and hyperopia) in phakic or aphakic persons with non-diseased eyes.

CONTRAINDICATIONS (REASON NOT TO USE)

DO NOT use miacare® DELIGHT Monthly Silicone Hydrogel Soft Contact Lens when any of the following exists:

- Inflammation or infection of the anterior chamber of the eye.
- Active disease, injury or abnormality affecting the cornea, conjunctiva, or eyelids.
- Microbial infection of the eye.
- Insufficiency of lacrimal secretion (dry eye) that interferes with contact lens wear Corneal hypoesthesia (reduced corneal sensitivity).
- Use of any medication that is contraindicated or interferes with contact lens wear, including eye medications.
- Any systemic disease which may be exacerbated by or interferes with contact lens wear.
- Allergic reactions or ocular irritation of the ocular surfaces or adnexa that may be caused by or exaggerated by the wearing of contact lenses.
- Allergy to an ingredient in a solution which must be used to care for the contact lenses.
- Patient history of recurring eye or eyelid infections, adverse effects associated with contact lens wear, intolerance or abnormal ocular response to contact lens wear.
- If eyes become red or irritated.

LENS WEAR SCHEDULE

The wearing and replacement schedule should be determined by the eye care professional.

BENQ materials recommends to wear in day time for 30 days, and it must be removed every night.

The lens are indicated for daily wear (less than 24 hours while awake).

LENS CARE DIRECTIONS

Patients must adhere to a recommended care regimen. Lenses must be cleaned, rinsed, and disinfected after removal and prior to reinsertion on the eye according to the instructions in the package inserts provided with the lens care products recommended by the eye care professional. Failure to follow the complete regimen in accordance with manufacturer's instructions in the package inserts may contribute to problems and/or result in the development of serious ocular complications as discussed in WARNINGS.

Replacement Wear:

When removed between replacement periods lenses must be cleaned and disinfected prior to reinsertion or be discarded and replaced with a fresh lens.

Basic Instructions for Lens Cleaning and Disinfection:

When lenses are dispensed, the eye care professional should recommend an appropriate system of lens care and provide the patient with instructions according to the package labeling.

WARNINGS

Advise patients of the following warnings pertaining to contact lens wear:

- Serious eye injury, scarring of the corneal, and loss of vision may result from problems associated with wearing contact lenses and using contact lens care products. To reduce these risks, emphasize to the patient the need for strictly compliance with the lens care regimen including hand washing, proper lens disinfection, cleaning of the lens case, wearing restrictions, wearing schedules, and follow-up visit schedules.
- Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision. Instruct patients at the dispensing visit and subsequent visits to immediately remove their lenses and promptly contact their eye care practitioner if they should experience eye discomfort, foreign body sensation, excessive tearing, vision changes, redness of the eye or other problems with their eyes.
- Non-compliance with the manufacturer's labeled lens care instructions may put the patient at significant risk of developing a serious eye infection.
- Tap water, distilled water, or homemade saline solution should NOT be used as a substitute for any component in the lens care process. The use of tap and distilled water has been associated with Acanthamoeba keratitis, a corneal infection that is resistant to treatment and cure.
- Smoking increases the risk of corneal ulcers for contact lens users, 3, 4 especially when lenses are worn overnight or while sleeping.
- The risk of microbial keratitis has been shown to be greater among users of extended wear contact lenses than among users of daily wear contact lenses. The risk increases with the number of consecutive days that the lenses are worn between removals, even with the first overnight use.

Visco Technology Sdn. Bhd.
2686, Jalan Todak, Seberang Jaya, 13700 Prai, Penang, Malaysia

Product by BenQ Materials Corp.
Address: 29 Jianquo E. Rd., Guishan, Taoyuan, Taiwan, R.O.C.

EC REP Emergo Europe B.V.
Prinsessegracht 20,2514 AP, The Hague, The Netherlands

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Produk oleh BenQ Materials Corp.